



## Lab Connection Instructions

### **PLANMECA**

Note: To send cases to Romexis LabApp you need to have a Romexis Cloud subscription with an adequate quota.

1. In the Romexis Cloud send window select the patient you would like to send and click on cloud export.
2. In the fillable "Send To" box type out our email, **scans@smiledesign.cloud**.
3. Select Send to another Planmeca Romexis option to send cases to Romexis LabApp.
4. Upload any scans and/or photos you would like to send.
5. Click on send and you are now finished. Our email should now be saved as a contact for you to be able to send future cases.