

Lab Connection Instructions



Medit:

1. Log in to www.meditlink.com with the administrator account.
2. Go to Partners on the left menu.
3. Search for SmileDesign by searching one of the following criteria below.
 - a. Lab Name - **SmileDesign**
 - b. Email address - **scans@smiledesign.cloud**
 - c. Address – **9069 W. Olive Ave, STE 111 PMB 418 Peoria, AZ 85345**
4. Locate the lab, and click the Request Partnership button. Check the partner's terms & conditions, and click the Confirm button.
5. You will see that the lab has been added to the Partners > My Partner List with the status reflected as Partnership Pending.
 - a. The "Partnership Pending" status will be changed to "Partnership" when the lab accepts the request.